

Langham Multi-Purpose Raiser

Raise Required	MPR Unit A	Spreader Link Bar E	Clip-on Height Section D
75mm (3")	2	1	0
95mm (3 ³ / ₄ ")	2	1	4
115mm (4 ¹ / ₂ ")	2	1	8
135mm (5 ¹ / ₄ ")	2	1	12
155mm (6")	2	1	16



• Raising Chairs

Raise Required	MPR Unit A	Spreader Link Bar E	Clip-on Height Section D
75mm (3")	2	1	0
95mm (3 ³ / ₄ ")	2	1	4
115mm (4 ¹ / ₂ ")	2	1	8
135mm (5 ¹ / ₄ ")	2	1	12
155mm (6")	2	1	16




• Raising Settees

*Small - 2 seater
with four castors**

*NB: The spreader link bar
used in the 2 seater settee
configuration should be the
LPMPR4 Extra Long Spreader
Link Bar.*


Raise Required	MPR Unit A	Spreader Link Bar E	Clip-on Height Section D
75mm (3")	3	0	0
95mm (3 ³ / ₄ ")	3	0	6
115mm (4 ¹ / ₂ ")	3	0	12
135mm (5 ¹ / ₄ ")	3	0	18
155mm (6")	3	0	24

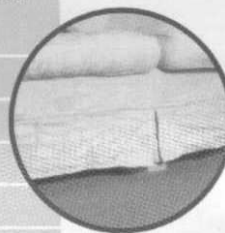

• Raising Settees

Large - 3 seater


Raise Required	MPR Unit A	Spreader Link Bar E	Clip-on Height Section D
75mm (3")	2	0	0
95mm (3 ³ / ₄ ")	2	0	4
115mm (4 ¹ / ₂ ")	2	0	8
135mm (5 ¹ / ₄ ")	2	0	12
155mm (6")	2	0	16

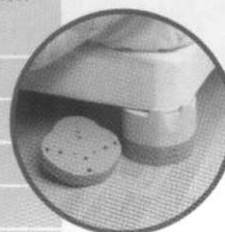

• Raising Single Beds

*With 4 Castors only**

Raise Required	MPR Unit A	Spreader Link Bar E	Clip-on Height Section D
75mm (3")	3	3	0
95mm (3 ³ / ₄ ")	3	3	6
115mm (4 ¹ / ₂ ")	3	3	12
135mm (5 ¹ / ₄ ")	3	3	18
155mm (6")	3	3	24

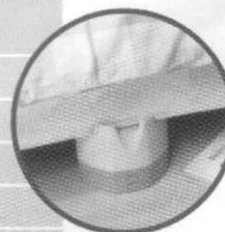

• Raising Double Beds

* If the settee has central castors then use the chart for large settees ** If the single bed has three sets of castors then use the double bed chart and omit the spreader link bars

PLEASE NOTE: These charts are a guide and you should always support all castors on a piece of furniture. Please refer to the user instructions or call our office if you have any questions regarding the safe use of the Multi-Purpose Raiser. It is possible to use the Extra Long Spreader Link Bar (LPMPR4) with the Multi-Purpose Raiser should you require an extra long spreader bar. For further information on Langham Products please telephone SmitCare.